



(/ 'yōonē,sōok/)

Patent Pending, 2022

UniSUC[®]

Unisex Siphoning Urinary Catheter

DIRECTIONS


Step

1

**~CLEAN PATIENT
~APPLY LABEL**



✓ Clean female or male perineal area

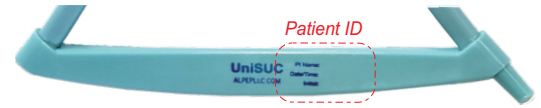

✓ Label Patient Name, Time & Initial

Step

2

BEND & FIT




✓ Rinse & dry Reusable Frame (if used)
✓ Bend Reusable Frame (cup facing body)
✓ Position bottom cup above anus
✓ Shape Frame to form body shape

Step

3

ASSEMBLE FRAME & LINER

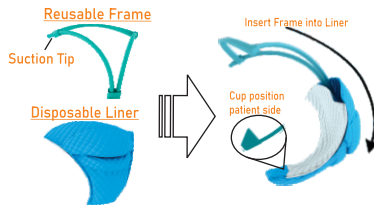

Combine UniSUC

✓ Insert Reusable Frame into Disposable Liner
✓ White fabric inner side & blue plastic outer side
✓ Face *cup* end toward patient
✓ Ensure *suction tip* is inserted through opening slot of Disposable Liner

Step

4

PLACEMENT & ATTACH HOSE




✓ Place Combined UniSUC onto patient
✓ Position above anus
✓ Ensure sides are contoured around edge of skin folds to patient comfort
✓ Connect the vacuum hose

Step

5

CONNECT TO VACUUM

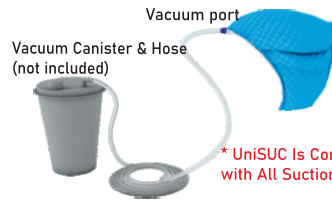

✓ Complete assembly of UniSUC with vacuum hose & canister
✓ Switch ON Suction Device for continuous suctioning (40mmHg)
✓ Replace Disposable Liner 8-12 hrs. (or if soiled)

HELPFUL TIPS

- ✓ Wear underwear over UniSUC for added support
- ✓ Patient can be turned q2h or perform ROM while wearing UniSUC
- ✓ Patient can wear UniSUC while on sitting position
- ✓ Follow Standard Nursing Procedure for urinal use & external catheters

Indications: UniSUC External Urinary Catheter is intended for non-invasive (external) urinary incontinence management for Male or Female patients. Latex free.

Contraindications: The UniSUC External Catheter should not be used on patients with urinary retention.

Warnings: This product is for external use only. Do not insert UniSUC Frame Device or its liners into the vagina, anal canal, or other body cavities. Avoid placement over skin wounds, surgical sites or hemorrhoidal tissue. Discontinue to use if an allergic reaction or bleeding occurs. After use, this product may be a potential biohazard. Dispose of in accordance with applicable local policies and regulations.



ALPEP LLC
1547 W. Struck Ave. G
Orange, CA 92867
www.alpepllc.com/unisuc
Toll free 877-709-0536
Local 562-474-6872
Fax 562-474-6872
Email: alpepllc@gmail.com



ALPEP, the ALPEP logo, UniSUC are trademarks of ALPEP LLC. All other trademarks are the property of their respective owners. © 2022 ALPEP LLC. All rights reserved.

alpepllc.com