



# WALL-MOUNT DUAL SUCTION CANISTER SETUP

CDC Guidelines. Refer to your facility standard guideline for more information

## Infection Control Guideline

- Use only closed suctioning system.
- Perform Hand hygiene
- Single-Patient use
- All tube lines & canisters to be replaced Q24H (or as per facility regulation)

## Prevent cross-contamination

- Suction tubing to be used separately for each apparatus
- Yankauer or inline tracheal suction tubes can be used jointly.
- Urine canister suction line to be on separate suction tube line.

## Department of Public Health Regulation for Urine Bag (Canister)

- Urine canister placed below bed and secured
- Urine canister to be draped or covered.

